# PILESEPTINE SPRAY

## EXTERNAL SPRAY

Medical device

**INDICATION:** Filmogen liquid dressing for external hemorrhoids

**PRESENTATION:** 30 ml aluminum container with sprayer.

**COMPOSITION:** Glycerol, Aqua, Pilecyanidin-E (extracts of *Vitis Vinifera* and *Vaccinium Macrocarpon*), *Mentha Piperita* essential oil.

**MODE OF ACTION:**

PILESEPTINE Spray is a viscous, osmotically active hypertonic solution. When applied on the surface of piles, the product forms a film over the hemorrhoids and mechanically attracts hypotonic fluid present below or around the affected surface. This strong outward flow of liquid from the tissue helps to reduce the edema and consequently the size of the hemorrhoid. The liquid outflow equally cleans the hemorrhoidal area free of contaminants (bacteria, cellular debris, dust, and other surface contaminants), and keeps the hemorrhoidal surface lastingly hydrated, which consequently reduces pain, irritation, itching, and burning sensation. Progressive reduction in the size of the hemorrhoid and a clean environment facilitate hemorrhoidal regression and support the natural cellular defenses, for a faster recovery. The film also protects the hemorrhoids against dryness, irritation and external aggressions.

**DIRECTIONS:** For topical application only, on external hemorrhoids. Clean the surface of hemorrhoids with a cotton gauze soaked in physiological saline or water. When the surface is dry, apply 3-4 sprays of PILESEPTINE solution directly onto the piles surface to form a thin product film over their entire area. Repeat the same procedure 3-4 times a day.

**PRECAUTIONS:** Wash hands before use. This product is for topical use only for external hemorrhoids. It should not be swallowed. Avoid contact between the tip of the spray and the piles during application. Wipe the tip of the spray with the help of a clean tissue paper after each use.

Do not use PILESEPTINE Spray if you are sensitive or allergic to any of the ingredients used to prepare this product, particularly if you have a known sensitivity to peppermint essential oil. Do not use PILESEPTINE Spray if you are using any other topical product for piles treatment. Seek medical advice if you see no improvement within 7-10 days. Keep out of reach of children. Store at room temperature, do not freeze or refrigerate. Do not use the product if the spray is damaged, after the expiry date or more than 30 days after 1<sup>st</sup> use.

**SIDE EFFECTS:** PILESEPTINE Spray has no known side effects and is not dangerous if swallowed. In certain cases, a slight local irritation may be observed but will disappear after a few minutes. PILESEPTINE Spray can be used in children above the age of 3 when applied by an adult. Safety of use in pregnant or lactating women has not been established. In absence of sufficient data, we do not recommend using this product during pregnancy or lactation.

**MANUFACTURER**  
VITROBIO  
ZAC de Lavaur  
63500 ISSOIRE, France

**DISTRIBUTOR**

MADE in FRANCE

COMMERCIAL NAMES: PILESEPTINE SPRAY.

In order to further improve the quality and efficacy of this product, we would greatly appreciate if you could complete and send us, along with your suggestions, the questionnaire on our website [www.vitrobio.com](http://www.vitrobio.com) (Questionnaire– Select Language– Piles).

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DRUGSSON AB
VITROBIO SAS